

## Performing Quality Control (QC) Tests

---

- Turn the Precision Xceed Pro on via the 'On/Off' button & select option '2' (2-Control Test) via the keypad
- When prompted Scan :
  - i. Your Operator Barcode
  - ii. The Low Level solution Barcode
  - iii. The Strip Lot Barcode
- Follow the steps on Screen
  - i. Before applying solution, gently invert to mix & prevent air locks
- Repeat the procedure for High Level Solution by :
  - i. Selecting option '1' (1-Next Level) via the keypad
  - ii. Following the on-screen prompts



## Performing Quality Control (QC) Tests

---

1. Open the strip using the triangular notch in the foil tearing down & across the length of the strip
2. Ensure the 3 electrodes enter the port facing upward
3. A sample can be applied to the top or tip of the test strip
4. Apply a sample with the meter in a horizontal (flat) position to avoid it entering the strip port
5. Fill trigger electrode ensures a sufficient sample is applied
6. The result, or 'PASS' or 'FAIL', is displayed after 20-second countdown

